



Drivers' Guide

Clinician's Home Visiting Case

This pictorial guide is for all CST and CST Support Assistants to aid you with the contents of the Badger clinician bags. Please ensure that all items are present and in full working order.

If any of the items pictured in the guide are missing or faulty report this immediately to the Team Leader who will be able to assist you with a replacement. Complete form CL-F-009 and return faulty equipment to the Meds. Team. Remember to also log this on the CST Shift Log and Check List stating whether a replacement was found.

Check expiry dates on all items, were necessary.

Contents Page

• Stethoscope	Page 3
• Thermoscan and thermoscan covers	Page 3
• Auroscope	Page 4
• (BP) Blood Pressure Machine & Cuff	Page 4
• Finger Pulse Oximeter	Page 5
• Blood Sugar Kit and Testing Strips	Page 5
• Patella Hammer	Page 6
• Lubricating Jelly	Page 6
• Sharps Yellow Box	Page 7
• Cotton Wool	Page 7
• Examination Gloves	Page 8
• Medical Hand Sanitiser	Page 8
• Pregnancy Testing Kit	Page 9
• Urine Dip Stick Bottle	Page 9
• GTN Spray	Page 10
• Emergency Red Box	Page 10
• AED (Defibrillator)	Page 11

Stethoscope



Thermoscan



Test thermoscan is working before undertaking visits. Change batteries (stored in CST room) if necessary. There should be enough thermoscan covers to complete all visits.

Auroscope



To switch on, to check to see if batteries need changing, turn the black dial at the bottom of the auroscope.

(BP) Blood Pressure Machine & Cuff



Finger Pulse Oximeter



Blood Sugar Kit & Test Strips




All BM kits are slightly different and may not be exactly as pictured. However, they should all include the blood sugar monitor, lancets, test strips and calibration solution.

Patella Hammer (Reflex Hammer)




Patella hammers may vary in size and shape to picture.

Lubricating Jelly

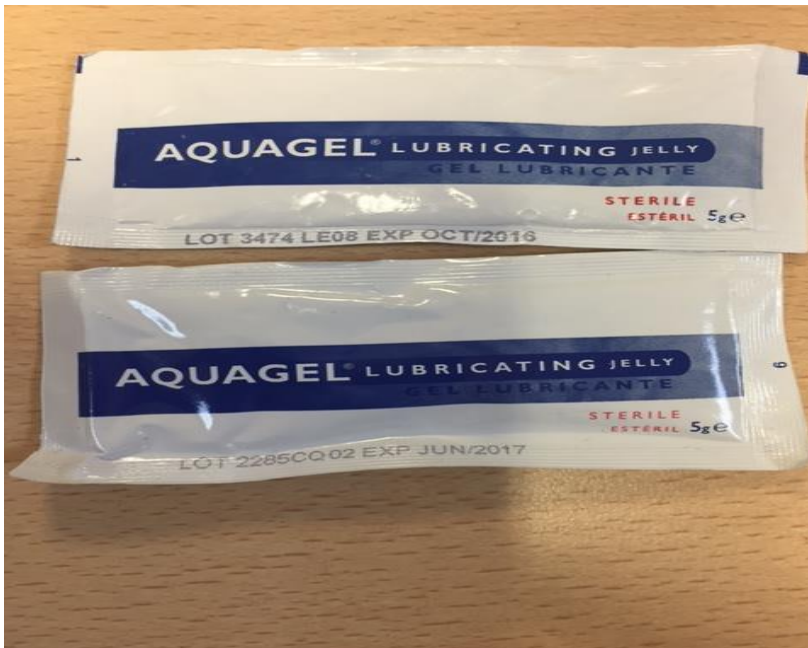

Lubricating Jelly is a water-based, water-soluble lubricant. Check relevant expiry dates.

Sharps Yellow Box

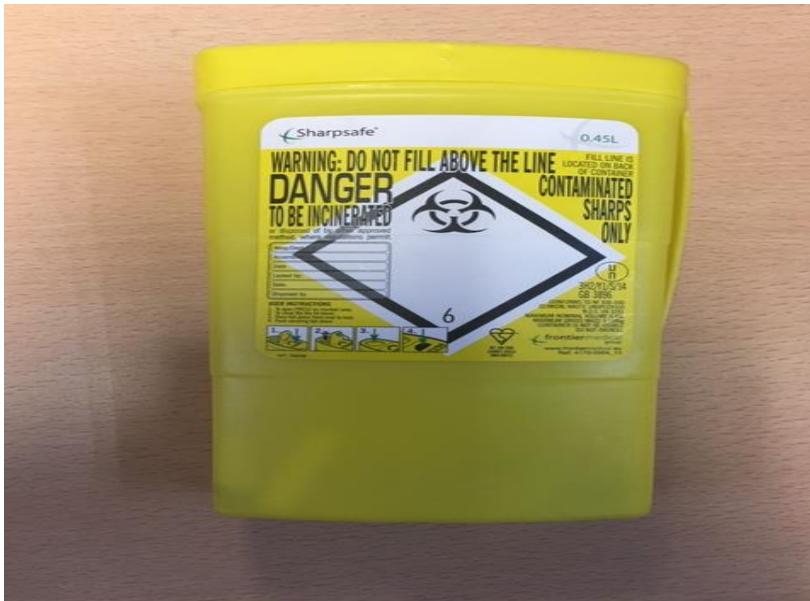

- All sharps, such as needles and lancets must be disposed of in a sharps bin.
- Do not place any other items in this container under any circumstances.
- Once the container is full, seal the lid and dispose in the clinical waste bin located outside of the drive through area of the Medical Centre at ABR. This should be done on return to site, following visits, before entering the drive through if necessary once bin is full.

Cotton Wool




Examination Gloves




Check that all sizes are available.

Medical Hand Sanitizer

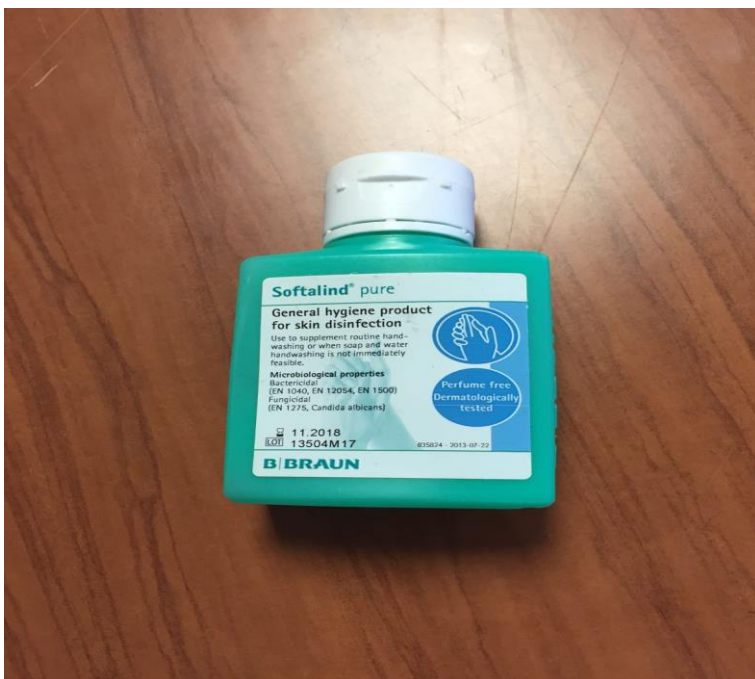

Pregnancy Testing Kit

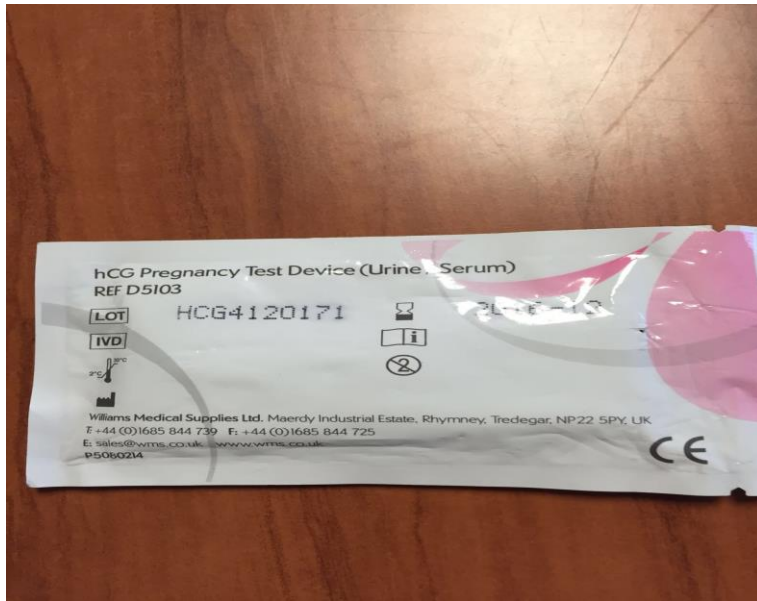

Check expiry dates before visits. Use the nearest to expiry first.

Urine Dip Stick Bottle



Check expiry dates before visits.

GTN Spray



Check expiry dates before visits.

Additional equipment located in Badger vehicles

Emergency Red Box



The Emergency Red Box is located within the boot of the Badger Vehicle. Make sure the box is sealed and within date at the start of each shift. It is important to record the boot temperature as this must be kept between 0c-25c.

AED (Defibrillator)



- This AED is located in the boot of the Badger vehicle.
- Check that the AED is in date (located of the display at the front of the defibrillator)
- Check for a green light at the top of the defibrillator. If a red light is displayed this must be taken out of circulation and replaced with another one. Report to the Team Leader. Complete CL-F-009 log on daily checking sheet.
- Check pads for AED are also in date.